

IO Test Conjugated Antibody CD127-APC

	Specifications
Specificity	CD127
Clone	R34.34
Hybridoma	NS1 x balb/c
Immunogen	Pre-Alp, pre-B leukemic cell line
Isotype	IgG1
Species	Mouse
Purification	Affinity chromatography
Fluorochrome	Allophycocyanin (APC)
Molar ratio	APC / Ig: 0.5 - 1.5
λ excitation	633/638 nm
Emission Peak	660 nm
Buffer	PBS pH 7.2 plus 2 mg / mL BSA and 0.1% NaN ₃

REF B42026 Liquid - 0.5 mL

Analyte Specific Reagent.

Analytical and performance characteristics are not established

REAGENTS

Concentration: See lot specific Certificate of Analysis at www.beckmancoulter.com.

WARNING AND PRECAUTIONS

1. This reagent contains 0.1% sodium azide. Sodium azide under acid conditions yields hydrazoic acid, an extremely toxic compound. Azide compounds should be flushed with running water while being discarded. These precautions are recommended to avoid deposits in metal piping in which explosive conditions can develop. If skin or eye contact occurs, wash excessively with water.
2. Specimens, samples and all material coming in contact with them should be considered potentially infectious and disposed of with proper precautions.
3. Never pipet by mouth and avoid contact of samples with skin and mucous membranes.
4. Do not use antibody beyond the expiration date on the label.
5. Do not expose reagents to strong light during storage or incubation.
6. Avoid microbial contamination of reagents or incorrect results might occur.
7. Use good laboratory practices when handling this reagent.
8. Any change in the physical appearance of the reagents may indicate deterioration and the reagent should not be used.

GHS HAZARD CLASSIFICATION

Not classified as hazardous

STORAGE AND HANDLING CONDITIONS AND STABILITY

This reagent is stable up to the expiration date when stored at 2 – 8°C. Do not freeze.

No reconstitution is necessary. This monoclonal antibody may be used directly from the vial. Bring reagent to 18 – 25°C prior to use.

CONTENTS

Sodium azide preservative may form explosive compounds in metal drain lines. See NIOSH Bulletin: Explosive Azide Hazard (8/16/76).

To avoid the possible build-up of azide compounds, flush wastepipes with water after the disposal of undiluted reagent. Sodium azide disposal must be in accordance with appropriate local regulations.

SPECIFICITY

CD127 is a member of the cytokine receptor family belonging to the Ig supergene family (1). CD127 contains is expressed as a 65-75 kDa and 90 kDa protein in B and T cells, respectively. The total molecular structure consists of 459 amino acids (extracellular 219, transmembrane 25 and intracellular 195 mono acids). MAb R34.34 was assigned to the CD127 cluster of differentiation at the 6th HLDA Workshop on Human Leucocyte Differentiation Antigens in Kobe, Japan, in 1996 (2).

TRADEMARKS

Beckman Coulter, the stylized logo and IOTest are trademarks of Beckman Coulter, Inc., and registered with the USPTO.

ADDITIONAL INFORMATION

For additional information, or if damaged product is received, call Beckman Coulter Customer Service at 800-526-7694 (USA or Canada) or contact your local Beckman Coulter Representative.

REFERENCES

1. Scott T. R. Walsh. Structural Insights into the Common ζ -Chain Family of Cytokines and Receptors from the Interleukin-7 Pathway.
2. Higuchi, M. Sugamura, K., "CD127Workshop Panel report", 1997, Leucocyte Typing VI, White Cell Differentiation Antigens. Kishimoto, T., et al., Eds., Garland Publishing, Inc., 838-840.



IMMUNOTECH S.A.S. a Beckman Coulter Company, 130, avenue de Lattre de Tassigny,

BP 177, 13276 Marseille cedex 9, France, 33-491 172 727